CLARIUM[®]

CLARIUM DERMAL FILLER

is available in three specialized models: SOFT, LIPS, and SHAPE, representing the next generation of skin fillers based on pure, non-animal hyaluronic acid. Designed using advanced BDDE Cross-Linking technology, these fillers provide an ideal balance of durability, tissue softness, and high safety. CLARIUM fillers are perfect for restoring lost volume, reducing wrinkles, and enhancing the natural contours of the face.

All models feature a consistent formulation of 20 mg/ml hyaluronic acid. By adjusting the degree of cross-linking and gel viscosity, each model is tailored for specific injection depths and treatment areas.

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CLARIUM DERMAL FILLER SOFT

Gentle and delicate for sensitive areas

- **Indication:** Ideal for under-eye hollows and superficial lines
- **Features:** Low viscosity, monophasic homogeneous structure for subtle filling and a natural, smooth appearance
- **Outcome:** Reduces dark circles and under-eye hollows, revitalizing and rejuvenating the skin
- **Duration:** 6–9 months

Scientific Parameters:

- **Injection Depth:** Superficial to mid-dermis
- **Needle Angle:** 10–15°
- **Needle/Cannula:** 30G/25G
- **Injection Technique:** Linear threading or micro-droplet
- **Volume per Point:** 0.5–0.75 ml per injection, depending on physician's assessment
- **Important Note:** Excessively deep injection may cause discoloration or edema (Tyndall Effect)

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CLARIUM DERMAL FILLER LIPS

Outcome:
Naturally full, well-shaped
lips with no heaviness,
preventing deep smile lines

Indication:
Lip augmentation,
contouring, hydration, and
smile line correction

Duration:
9–12 months

Features:
Medium viscosity, uniform
monophasic structure, high
resistance to repetitive

Scientific Parameters:

- **Injection Depth:** Mid-dermis
- **Needle Angle:** 30–40° for lip body / 10–15° for border definition
- **Needle/Cannula:** 30G for lips, 27G for smile lines/25G
- **Injection Technique:** Linear threading, Fanning, or Serial puncture
- **Volume per Lip:** 0.5–1.0 ml per session, depending on anatomy
- **Important Note:** Apply consistent pressure to avoid vascular compromise

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Professional contouring and volumizing

- **Indication:** Volumizing and correction of deep lines, contouring of cheeks, chin, jawline, and nose shaping
- **Features:** High-viscosity gel suitable for lifting, volumizing, and long-lasting facial contour
- **Outcome:** Restores lost volume and defines facial features with long-term results
- **Duration:** 9–12 months

Scientific Parameters:

- **Injection Depth:** Deep dermis to subdermal plane, occasionally periosteal
- **Needle Angle:** 45°
- **Needle/Cannula:** 25G & 27G / 23G
- **Injection Technique:** Depot (Bolus) for volumizing points or Linear threading for contouring
- **Volume per Point:** Depends on physician's assessment and patient anatomy
- **Important Note:** Inject slowly to prevent intravascular injection. Aspiration before injection is recommended.

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- Elasticity (G')
- Viscosity
- Spreadability

Rheological Profile (Conceptual)

- SOFT: Low G' → High adaptability in superficial tissue
- LIPS: Balanced G' → Dynamic volume & flexibility
- SHAPE: High G' → Projection, lift and support

Soft


Lips

Shape


Rheology Diagrams for CLARIUM DERMAL FILLER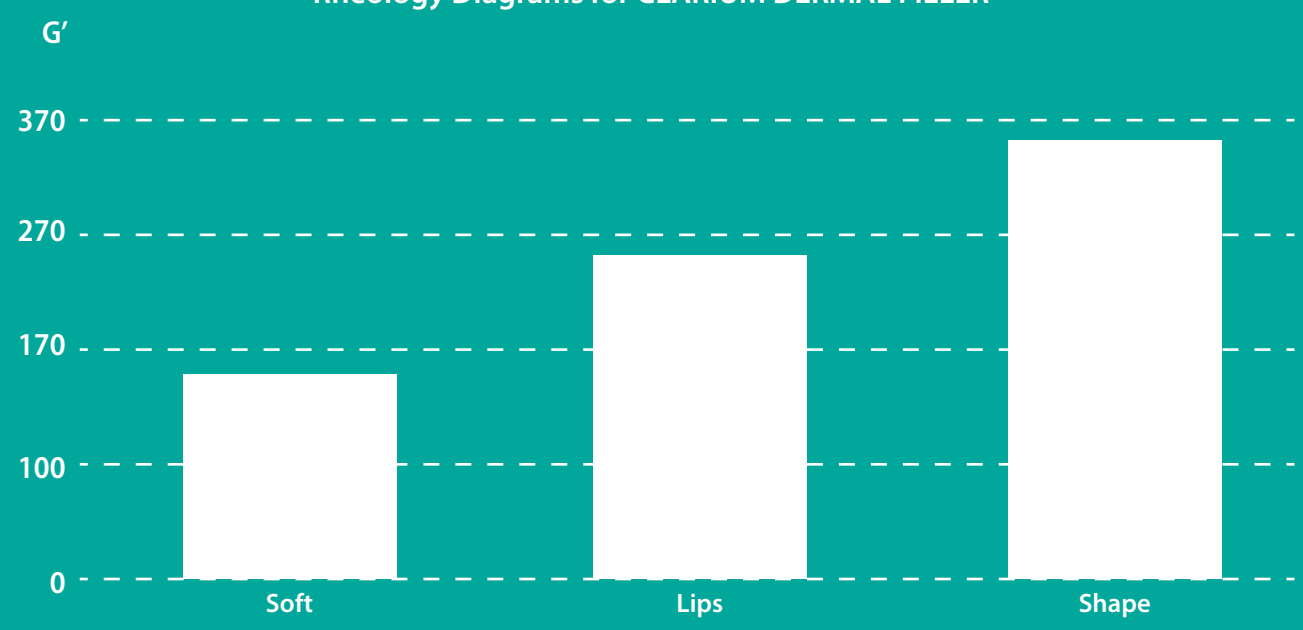


RHEOLOGY DIAGRAM


CLARIUM DERMAL FILLER SOFT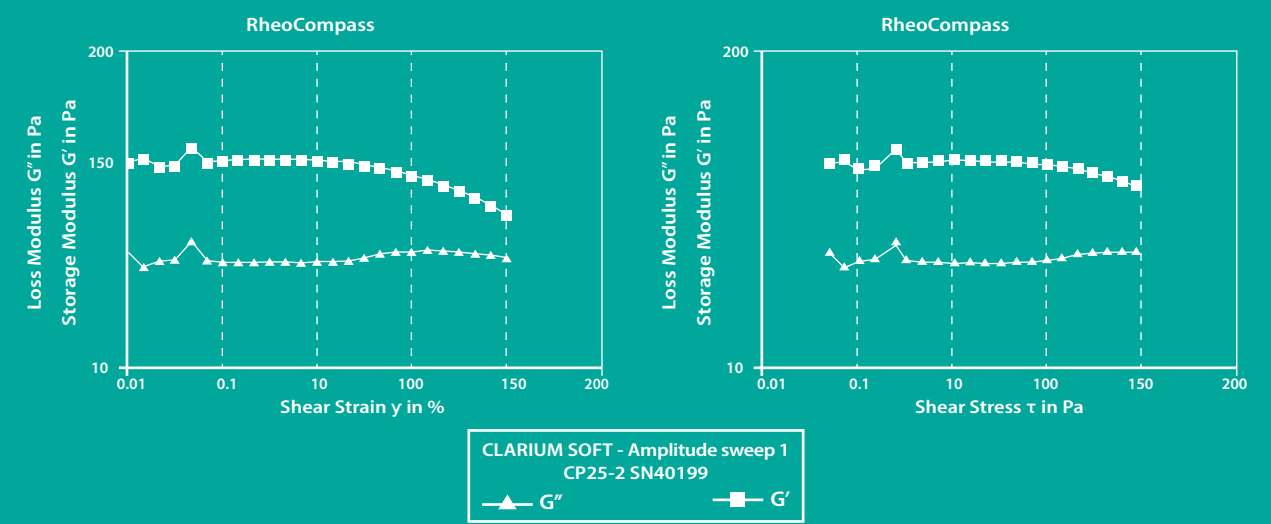

CLARIUM DERMAL FILLER LIPS

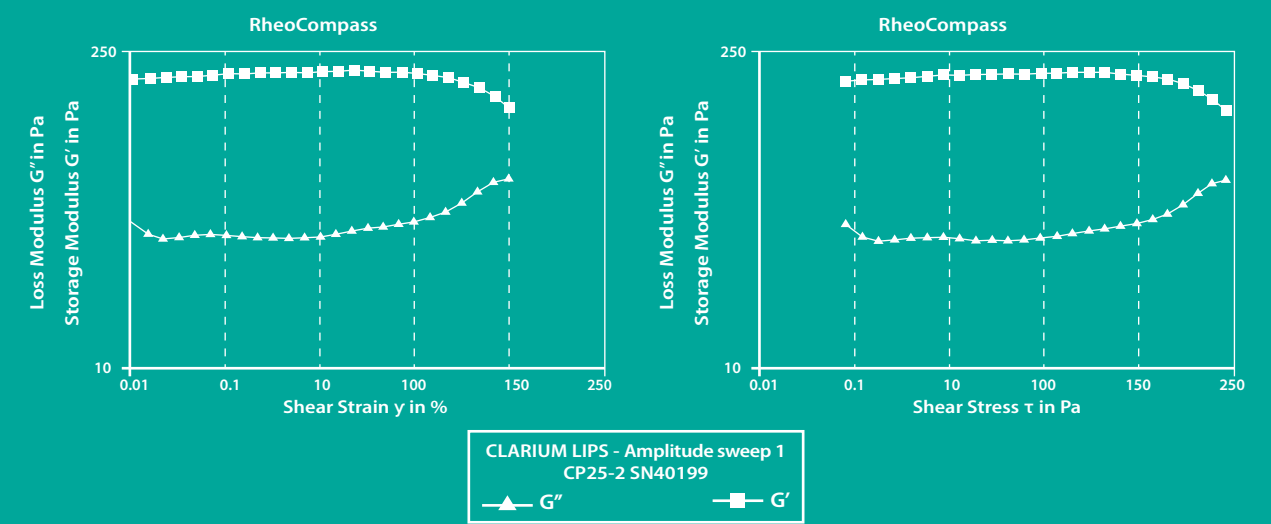CLARIUM DERMAL FILLER SHAPE

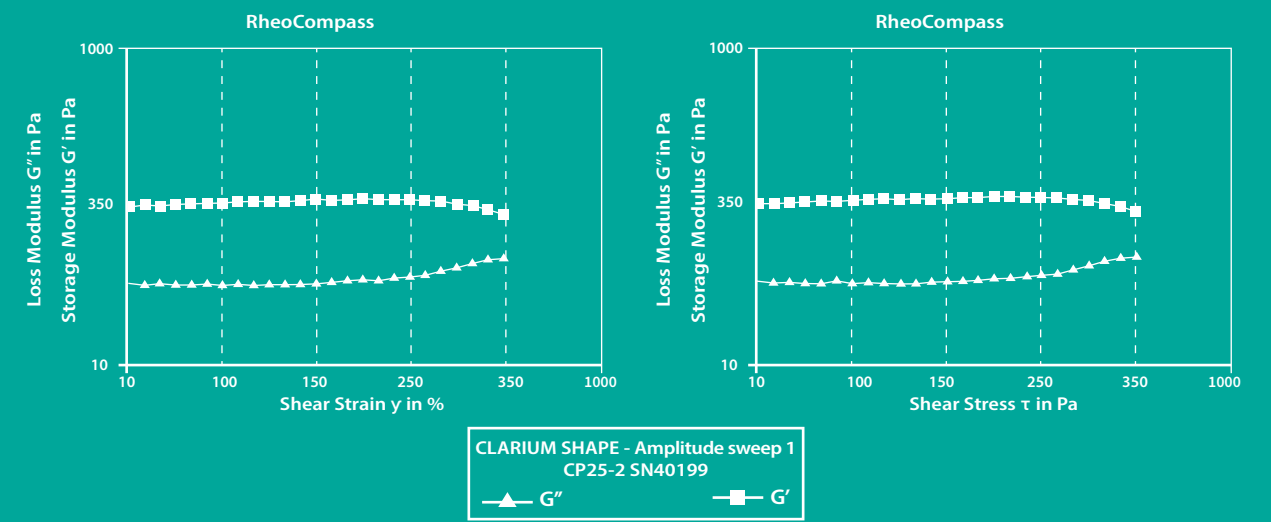
			
Feature	SOFT	LIPS	SHAPE
Main Indication	Under-eye hollows & fine lines	Lips augmentation & smile lines	Cheek, chin, jawline contouring & nose shaping
HA Concentration	20 mg/ml	20 mg/ml	20 mg/ml
Cross-Linking (BDDE)	Low	Medium	High
Viscosity	350,00-500,000 cp	450,00-650,000 cp	550,00-750,000 cp
Recommended Injection Depth	Superficial to midermis	Mid-dermis	Deep dermis to subdermal
Particle Size	100-200 µm	280-400 µm	350-550 µm
Average Duration	6-9 months	9-12 months	9-12 months
Product Volume	1ml	1 ml	1ml , 3 ml
Recommended Needle	30G	27G, 30G	27G, 25G
Cannula	25G	25G	23G

CLARIUM

DERMAL
FILLER

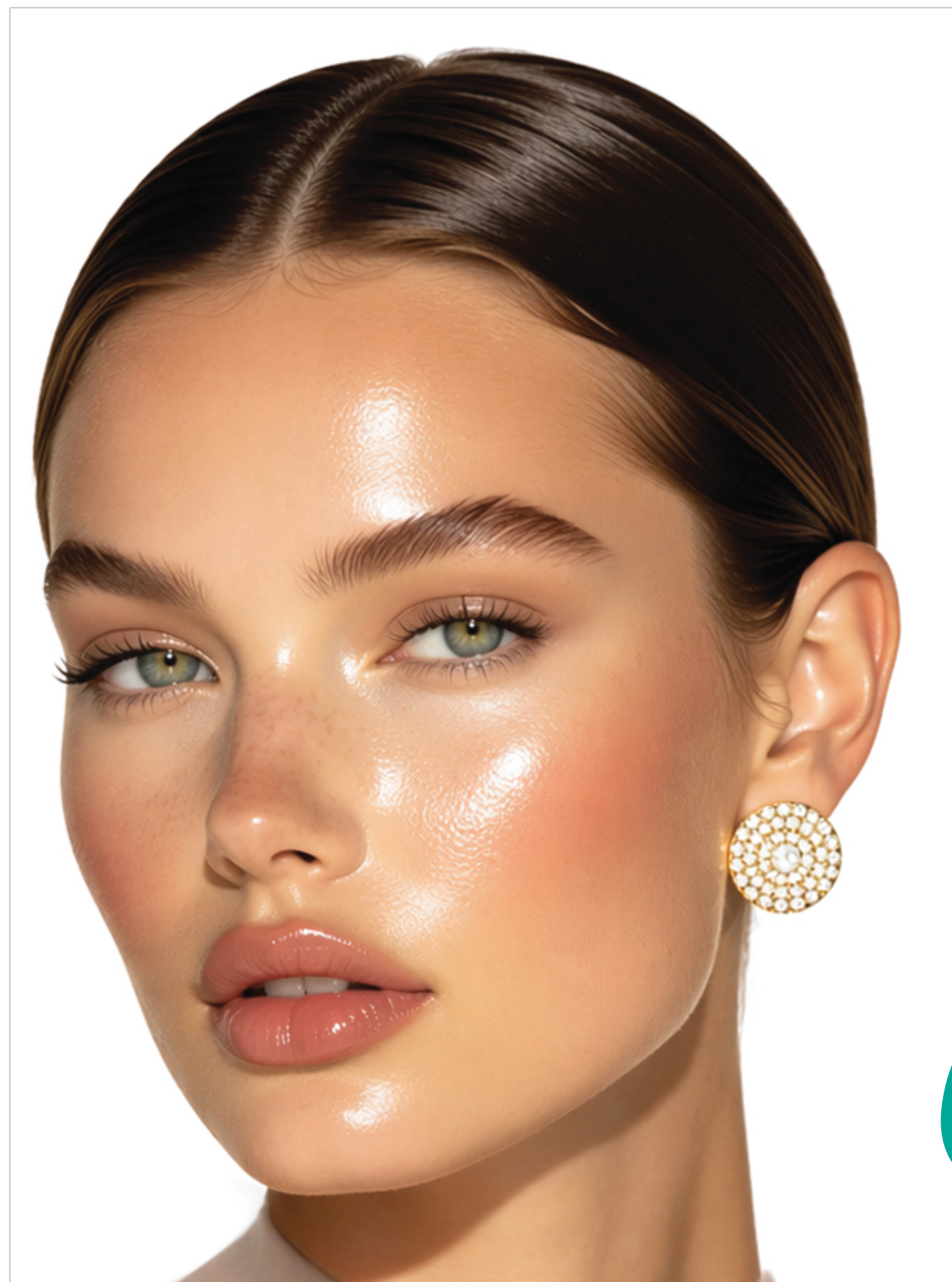C

SCAN ME

WWW.CLARIUM.SKIN